

INFUSION/BLOOD WARMER

ANIMEC AM-301

RELIABLE, SAFE, COST-EFFECTIVE

NO DISPOSABLES

ADJUSTABLE TEMPERATURE

TEMPERATURE DISPLAY

OVER-TEMPERATURE ALARM

SIMPLE OPERATION

With the ANIMEC AM-301, warmed fluid can be efficiently delivered to patients with ease, safety and no risk of contamination.

The ANIMEC AM-301 will surely benefit your patients and hospital.



Hypothermia, which may result in serious side-effects, has always been a major concern. Infusion/Transfusion at body temperature helps prevent and treat hypothermia and also assists effective intravenous administration.

The **ANIMEC AM-301** has various special features to

- **meet individual needs**

User-adjustable set-point temperature
[36°C or 39°C]

- **ensure safe use**

Easy to read LED display of actual outlet fluid temperature
Audible and Visible over-temperature alarms
Alarm test available by the "check" button

- **save cost**

No disposable set required
Available for two standard tube sizes
(4mm & 5mm)

- **extend application**

User-selectable flow rate range [gravity drip]
1-15ml/min: Single-channel setting [photo1]
1-20ml/min: Dual-channel setting [photo2]

- **offer greater convenience**

Mountable horizontally and vertically
Compact and light-weight with a handle
Easy maintenance



"Simple operation/Quick warm up"

1. Set a tube into the channel and fill it up with fluid.
2. Turn on the power.
3. Adjust set-point temperature and push the "start" button.

APPLICATIONS

Infusion or Transfusion for pre-surgery,
post-surgery or short-term surgery

SPECIFICATIONS

Type	AM-301-4A/5A	AM-301-4B/5B
Tube Size	4A: 3.1-4.0mm ϕ 5A: 4.1-5.0mm ϕ	4B: 3.1-4.0mm ϕ 5B: 4.1-5.0mm ϕ
Flow Rate	1 – 20 ml/min	
Set-point Temperature	36°C / 39°C	
Alarm Temperature	42.5°C (Outlet Fluid Temperature)	
Heat Exchanger	Dry Heat [Silicone Rubber Heater]	
Power Requirements	220-250V~ 50/60Hz	110-120V~ 50/60Hz
Dimensions	L 115 x W 65 x H 265 (mm)	
Weight	1 kg	

CLICK HERE

To Go Back To Paragon Medical